



**The Silentia
Screen System**





Keep it *Clean*[™] for a safer health care setting

At Silentia, we partner with health care facilities around the world to deliver a privacy screen system that offers safety and optimal patient care. Keep it *Clean* is our commitment to raising the bar for improved hygiene standards and infection prevention. Thanks to the EasyCare functions, our screens are easy to disinfect, move and combine. Our products consist of different lengths and heights, all part of a complete system that can adapt to your specific needs.

EasyClean[™] for optimal hygiene

The screens have hard and smooth surfaces that withstand all disinfectants used in patient care settings today. The wheels can easily be removed and washed separately in a dishwasher. All products meet strict hygiene requirements and all surfaces are accessible for cleaning – that's EasyClean.

Folds back easily with one hand – EasyReturn[™]

The patented hinges have a built-in memory which enables easy handling and prevention of cross infection using only one contact point: the ergonomically designed handle.

Fixed or mobile – the choice is yours with EasyClick[™]


All of our screens are equipped with the EasyClick function making mobility one click away. A wall-mounted screen can be mounted on a trolley for mobile use or to any of our system's other components.



EasyClean



EasyClean

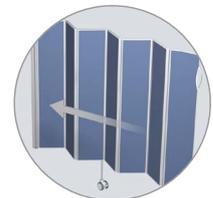

EasyReturn



EasyClick



Floor poles allow you to create a service corridor behind the head of the bed




Folding screens

Simplicity is one of our guiding principles in product development. It has to be easy to screen off patients when they need privacy or simply want time to themselves. It also has to be easy to change things around and expand the system – so all components are designed to be compatible with each other.


WALL INSTALLATION
This standard form of installation enables a screen to be attached directly to the wall.

The 25 cm panel width saves you valuable wall space without sacrificing patient privacy.

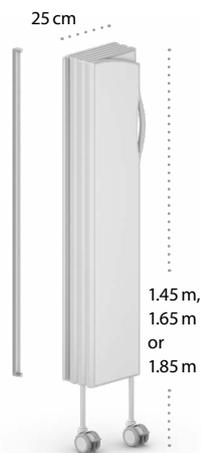

SINGLE TROLLEY
Accessory used to make folding screens mobile.

The wheelbase is designed to take up a minimal amount of space.

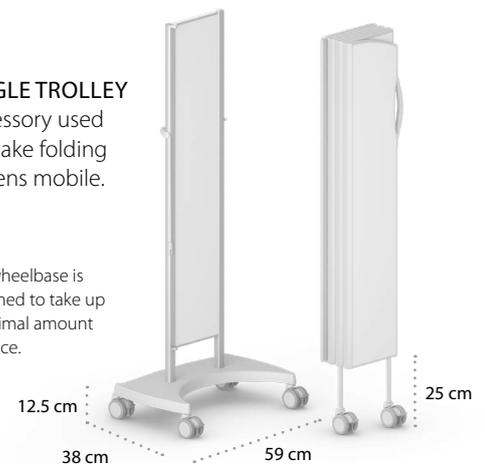


POLE FOR FLOOR INSTALLATION

Designed to be anchored to the floor. This is applied when a free-standing screen is required.




WALL INSTALLATION WITH SPACER

Used where a screen has to be installed a short distance from the wall, to provide clearance for a care panel, for example.

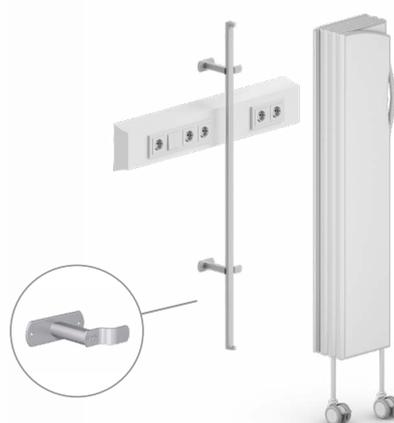

INSTALLED ON WALL RAIL

Clamps are used when a screen has to be installed directly to the wall rail.

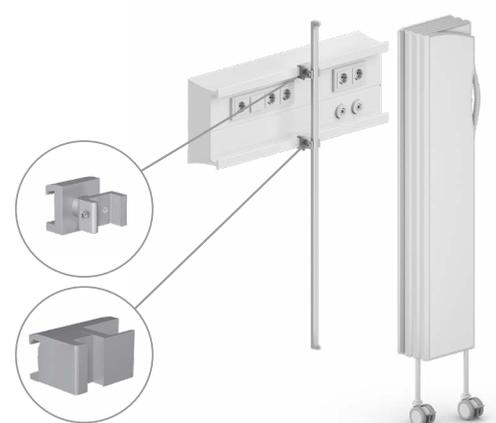



Mobile single screens

MOUNTED ON TROLLEY

The screen can be mounted on a trolley to provide mobile privacy when necessary. This trolley is intended for folding screens in various lengths: from 5 to 15 sections. Height: 1.45 m, 1.65 m or 1.85 m. The wheelbase takes up a minimal amount of floor space, while providing maximum stability.






Mobile double screens

MOUNTED ON TROLLEY

Our mobile double is a free-standing screen with two folding screens mounted back to back on a trolley. A smart and versatile solution for maximum flexibility and patient privacy. Takes up minimal space when not in use.

The mobile double screen can be used separately at the end of the bed, in an L- or U-shape, to create a movable and flexible space in the room.


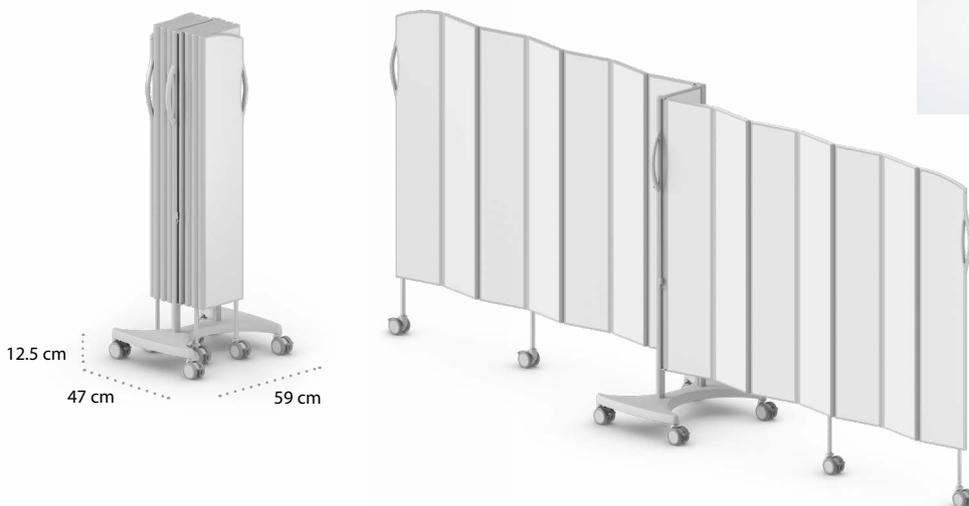




Mobile bed end screens

SILENTIA BED END SCREEN 1.5 M



Provides optimal privacy at the end of the bed – or for changing. The wheelbase takes up a minimal amount of floor space, while providing maximum stability. Extended width: 1.5 m. Height: 1.55 m or 1.85 m.

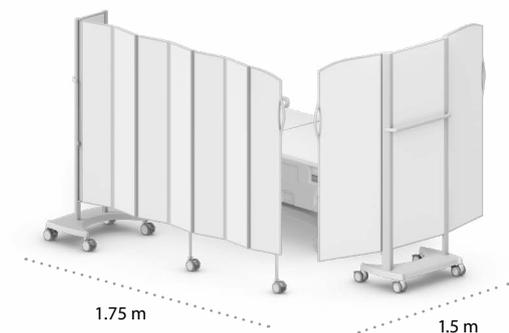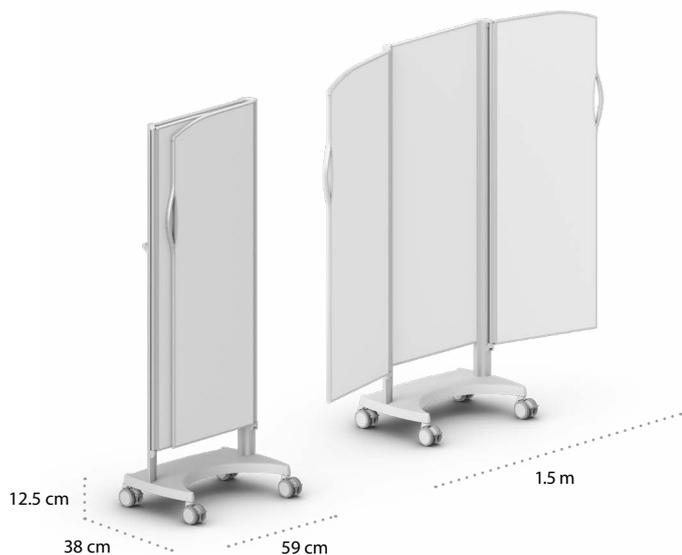





SILENTIA BED END SCREEN 2.5 M

This bed end screen is ideal when a wider solution is required.
 Extended width: 2.5 m. Height: 1.55 m or 1.85 m.

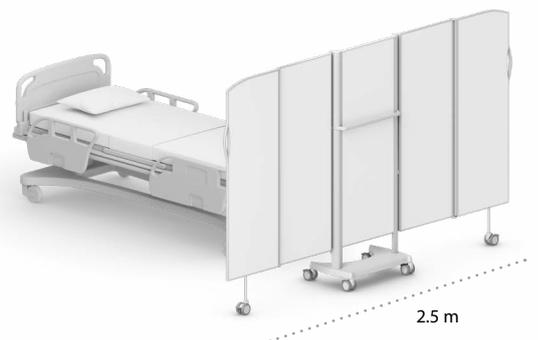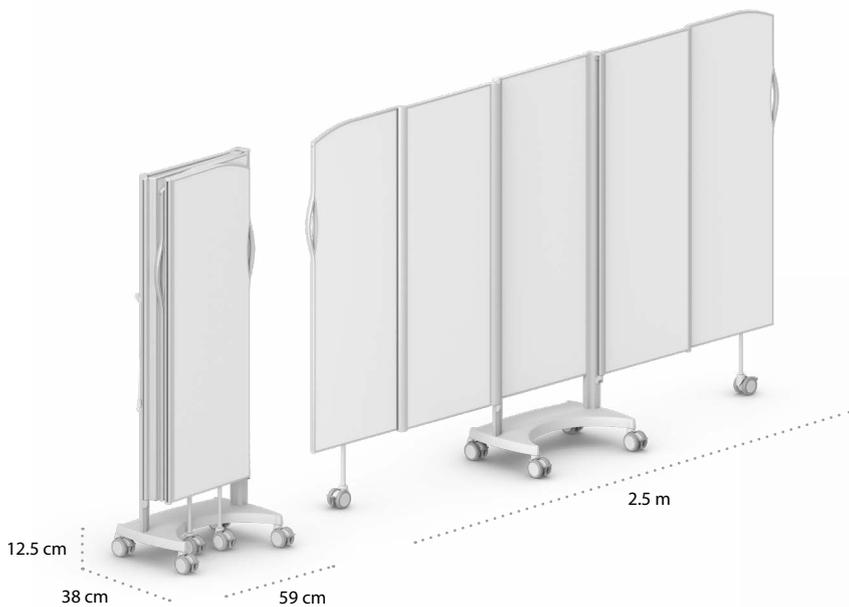

Folding screen technical specifications

STANDARD SIZES

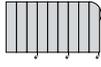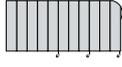
| | NO. OF PANELS | ART. NO. COLOR | ART. NO. DAYLIGHT TOP | ART. NO. DAYLIGHT DUO | ART. NO. DAYLIGHT | LENGTH | HEIGHT | WEIGHT | FOLDED SIZE | |
|---|---------------|----------------|-----------------------|-----------------------|-------------------|--------|--------|---------|-------------|--------|
| | | | | | | | | | WIDTH | DEPTH |
|  | 5 | 06105 | 06105-DT | 06105-DD | 06105-D | 1.25 m | 1.45 m | 8.0 kg | 0.25 m | 0.10 m |
| | | 06205 | 06205-DT | 06205-DD | 06205-D | 1.25 m | 1.65 m | 9.5 kg | 0.25 m | 0.10 m |
| | | 06305 | 06305-DT | 06305-DD | 06305-D | 1.25 m | 1.85 m | 11.5 kg | 0.25 m | 0.10 m |
|  | 7 | 06107 | 06107-DT | 06107-DD | 06107-D | 1.75 m | 1.45 m | 12.0 kg | 0.25 m | 0.14 m |
| | | 06207 | 06207-DT | 06207-DD | 06207-D | 1.75 m | 1.65 m | 14.0 kg | 0.25 m | 0.14 m |
| | | 06307 | 06307-DT | 06307-DD | 06307-D | 1.75 m | 1.85 m | 16.0 kg | 0.25 m | 0.14 m |
|  | 9 | 06109 | 06109-DT | 06109-DD | 06109-D | 2.25 m | 1.45 m | 15.5 kg | 0.25 m | 0.18 m |
| | | 06209 | 06209-DT | 06209-DD | 06209-D | 2.25 m | 1.65 m | 18.0 kg | 0.25 m | 0.18 m |
| | | 06309 | 06309-DT | 06309-DD | 06309-D | 2.25 m | 1.85 m | 20.5 kg | 0.25 m | 0.18 m |
|  | 11 | 06111 | 06111-DT | 06111-DD | 06111-D | 2.75 m | 1.45 m | 18.5 kg | 0.25 m | 0.22 m |
| | | 06211 | 06211-DT | 06211-DD | 06211-D | 2.75 m | 1.65 m | 21.5 kg | 0.25 m | 0.22 m |
| | | 06311 | 06311-DT | 06311-DD | 06311-D | 2.75 m | 1.85 m | 24.5 kg | 0.25 m | 0.22 m |
|  | 13 | 06113 | 06113-DT | 06113-DD | Not available | 3.25 m | 1.45 m | 22.0 kg | 0.25 m | 0.26 m |
| | | 06213 | 06213-DT | 06213-DD | | 3.25 m | 1.65 m | 25.5 kg | 0.25 m | 0.26 m |
| | | 06313 | 06313-DT | 06313-DD | | 3.25 m | 1.85 m | 29.0 kg | 0.25 m | 0.26 m |
|  | 15 | 06115 | 06115-DT | 06115-DD | Not available | 3.75 m | 1.45 m | 25.5 kg | 0.25 m | 0.30 m |
| | | 06215 | 06215-DT | 06215-DD | | 3.75 m | 1.65 m | 29.5 kg | 0.25 m | 0.30 m |
| | | 06315 | 06315-DT | 06315-DD | | 3.75 m | 1.85 m | 33.5 kg | 0.25 m | 0.30 m |

The table shows our standard sizes in 0.5 m length increments. Screens are also available in increments of 0.25 m if the total length is between 0.75 m and 3.75 m. The standard heights – 1.45 m, 1.65 m and 1.85 m – can be lowered by –0.1 m by determining the height of the legs of the wheels. The height is decided when you order your Silentia folding screen. **Note:** For maximum stability, the longest screens (12–15 panels) are intended for use in an L-shape for a single trolley and a U-shape for a double trolley around the bed.




COLOR SCREENS

Choose your screen color or design on page 19.

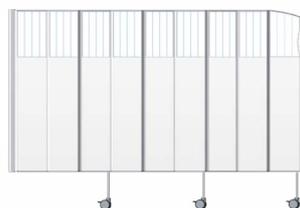


DAYLIGHT TOP

Daylight top panel is about 0.3–0.4 m high. Depends on which panel height you choose.

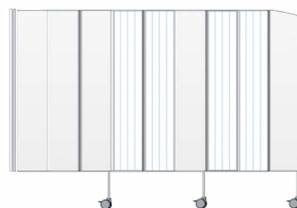

DAYLIGHT DUO

Frosted panels come in pairs.

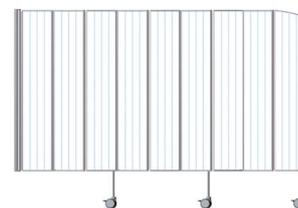


DAYLIGHT



CHOOSE THE RIGHT WALL MOUNTING TO SUIT DOOR LOCATION

(A) The folding screen mounting is normally on the left side, which means the screen is folded to the right of the wall mounting.

(B) To prevent the screen from taking up too much space next to a doorway for example, screens can also be ordered with the mounting on the opposite side.

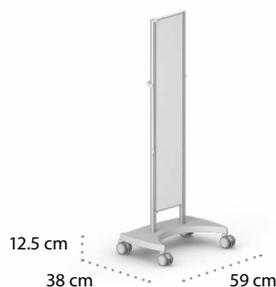

SINGLE TROLLEY FOR ONE SCREEN


Art. no. 06514, height 1.45 m

Art. no. 06518, height 1.65 m

Art. no. 06520, height 1.85 m

Accessory for folding screens up to 15-panels long. EasyClick screen mounting.

Note: The longest panels (12–15 sections) are intended for use in an L-shape around the bed.

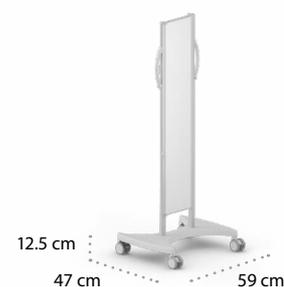

DOUBLE TROLLEY FOR TWO SCREENS


Art. no. 06501, height 1.45 m

Art. no. 06503, height 1.65 m

Art. no. 06521, height 1.85 m

Accessory for folding screens up to 15-panels long. EasyClick screen mounting.


Note: The longest panels (12–15 sections) are intended for use in a U-shape around the bed.

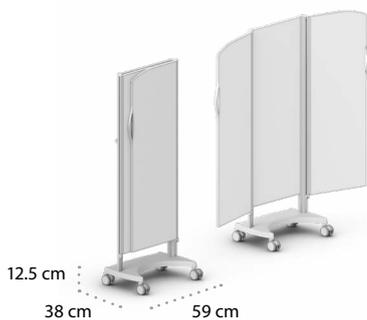

SILENTIA BED END SCREEN 1.5 M

Art. no. 06522, height 1.55 m
 Art. no. 06523, height 1.85 m

Extended width 1.5 m, folded width 0.55 m. This screen is intended for situations where a mobile screen is needed, e.g. changing or at the end. The small wheelbase saves valuable floor space and its low height reduces the risk of tripping over it.

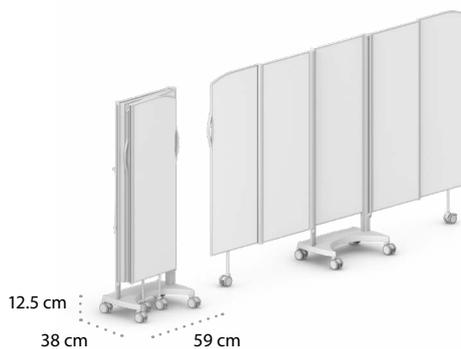

SILENTIA BED END SCREEN 2.5 M

Art. no. 06532, height 1.55 m
 Art. no. 06533, height 1.85 m

Extended width 2.5 m, folded width 0.55 m. Provides effective privacy at the end of the bed, in hallways or for changing. The wheels can be locked when the screen is extended. The small wheelbase saves valuable floor space and its low height reduces the risk of tripping over it.



COLORS AND DESIGN

Choose your screen color or design on page 19.



Fixed screens


Our fixed screens have been developed for applications where more permanent privacy is required. This system also allows a wider range of possibilities. For example, a fixed screen between beds can be combined with a folding screen at the foot of the bed. Fixed screens can be used to create changing cubicles and other types of spaces within a room, without making alterations to walls.

STANDARD SIZES

| LENGTH | HEIGHT | WEIGHT | ART. NO. COLOR | ART. NO. DAYLIGHT TOP | ART. NO. DAYLIGHT |
|--------|--------|---------|----------------|-----------------------|-------------------|
| 1.3m | 1.45 m | 6.2 kg | 06150 | 06150-DT | 06150-D |
| 1.3m | 1.65 m | 7.2 kg | 06250 | 06250-DT | 06250-D |
| 1.3m | 1.85 m | 8.3 kg | 06350 | 06350-DT | 06350-D |
| 2.4m | 1.45 m | 11.4 kg | 06155 | 06155-DT | 06155-D |
| 2.4m | 1.65 m | 13.3 kg | 06255 | 06255-DT | 06255-D |
| 2.4m | 1.85 m | 15.3 kg | 06355 | 06355-DT | 06355-D |




DAYLIGHT, FIXED

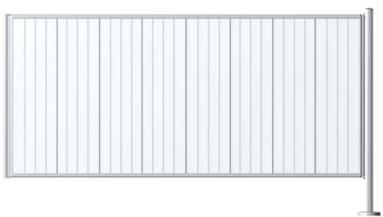

COLOR SCREEN, FIXED



DAYLIGHT TOP, FIXED

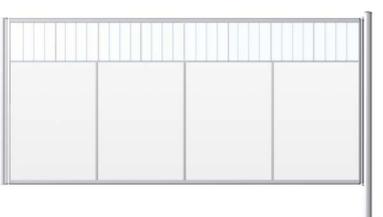



FLOOR INSTALLATION


A standard floor pole (A) is used to install a single fixed screen with EasyClick. If you want the pole to serve as a mounting point for additional fixed or folding screens, there are special versions (B–D) with up to four mounting points.

The pole is normally secured to the floor for maximum stability – this is especially important if the pole supports more than one screen. If the pole is only intended to support a single wall-mounted screen, it can simply rest on the floor.

STANDARD SIZES FLOOR POLES

| HEIGHT | WEIGHT | ART. NO. | ART. NO. | ART. NO. | ART. NO. |
|--------|--------|----------|----------|----------|----------|
| 1.45 m | 3.3 kg | 06561-A | 06561-B | 06561-C | 06561-D |
| 1.65 m | 3.5 kg | 06562-A | 06562-B | 06562-C | 06562-D |
| 1.85 m | 3.8 kg | 06563-A | 06563-B | 06563-C | 06563-D |

Floor pole attachment possibilities – seen from above.

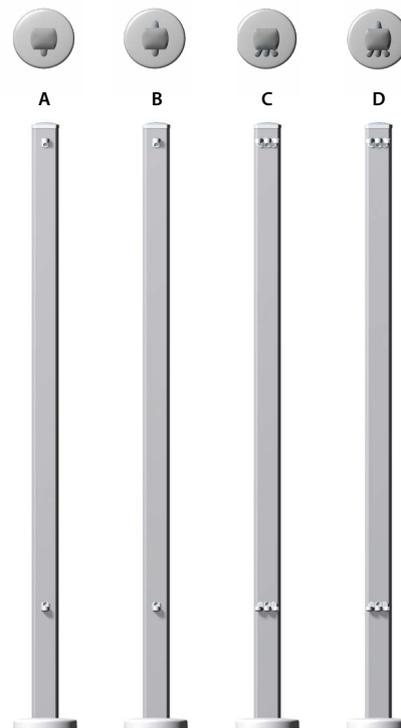

The pole and cover of the pole at the base are attractively designed with a smooth surface that is easy to clean.



EXTENSION SCREEN

Moves the mounting point for the folding screen away from the wall, either to make space for the bedside table or to extend a folding screen. Available in four lengths and three heights.

| HEIGHT | ART. NO. LENGTH 0.5 M | ART. NO. LENGTH 0.75 M | ART. NO. LENGTH 1.0 M | ART. NO. LENGTH 1.2 M |
|--------|--------------------------|---------------------------|--------------------------|--------------------------|
| 1.45 m | 06162 | 06163 | 06164 | 06165 |
| 1.65 m | 06262 | 06263 | 06264 | 06265 |
| 1.85 m | 06362 | 06363 | 06364 | 06365 |



ClearPanels™

ClearPanels is a completely transparent, mobile screen that makes it easy to monitor and communicate with patients. It meets the health care sector's strict hygiene requirements with smooth and hard surfaces that are easily accessible and withstand disinfectants. It's easy to clean, move and store when not in use.


STANDARD SIZES

The wheelbase takes up a minimal amount of floor space, while providing maximum stability.

- The lightweight construction is user-friendly and optimizes stability and safety.
- Section panels are made of polycarbonate that withstand disinfectants used in any patient care setting.



ClearPanels is available with a split panel for spirometry tests and examination. Art no. 06523-CP-S.

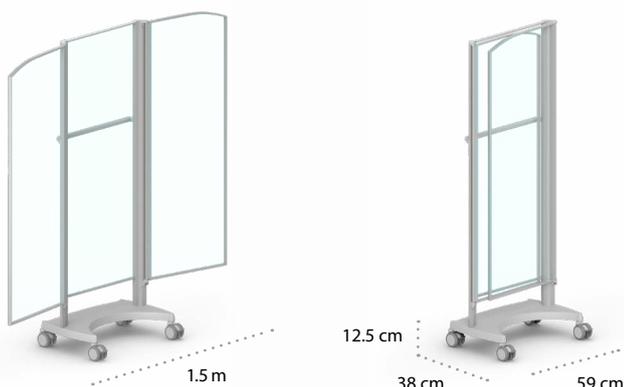

| ART. NO. | LENGTH | HEIGHT | WEIGHT | FOLDED SIZE | |
|------------|--------|--------|---------|-------------|--------|
| | | | | DEPTH | WIDTH |
| 06522-CP | 1.5 m | 1.55 m | 27.5 kg | 0.38 m | 0.55 m |
| 06523-CP | 1.5 m | 1.85 m | 30.5 kg | 0.38 m | 0.55 m |
| 06523-CP-S | 1.5 m | 1.85 m | 30.5 kg | 0.38 m | 0.55 m |



Shower screens

The Silentia shower screen makes it easy to provide patient privacy. The lowest height allows the caregiver to shower a patient sitting on a stool while keeping the caregiver dry. The wheels have a spring suspension providing smooth handling even on uneven floors. It easily folds back to the wall when not in use.

The wall-mounted screen is available in 3 different lengths and heights. EasyClick makes it simple to remove the screen for cleaning of the shower area. Even the wheels are easily detached thanks to EasyClick.



Double Left



Single



Double Right





StoryPanels™

A picture is worth a thousand words and can even have a healing impact. Silentia's StoryPanels offer a new way to give children a playful and positive atmosphere by creating a positive distraction. A child in a hospital is far from home and among unfamiliar faces, but by engaging with artwork such as planets and characters the child immediately starts to use its fantasy and imagination.

ArtPanels™

Nature provides tranquility in a complex world. Research shows that art with views of nature and animal life has a positive effect on hospital patients' recovery. With the bright colors of nature, ArtPanels is designed to give hospital patients a stress-reducing effect by letting them focus on something positive instead of their condition.



PhotoPanels™

Silentia's PhotoPanels help create privacy, peace of mind and a comfortable feel. Studies show that images and colors can have a positive influence on people by lowering their blood pressure, stimulating positive thoughts and relieving stress, pain and anxiety. This knowledge is ideal to explore in elder care, hospitals and other healthcare settings.

RELAXING, REMEMBERING, PROTECTING

PhotoPanels has been developed as a professional helping tool to stimulate people in different situations by creating peace and calm, security and protection or memories of recognizable places. At the same time, they offer optimal hygiene and privacy without feeling closed in.

CUSTOMIZE WITH YOUR LOGO, COLORS OR MESSAGES

Show your visual identity on your folding screens and create familiarity with your brand. When using your logo consistently, it becomes a strong, recognizable and trustworthy symbol of your institution and employees. You can add identification text on the screens, such as department, section, area, etc. You can also add short messages, directions or important notifications. You can also add short messages on the screens for guidance or important messages.

Installation

Clamps



STANDARD CLAMP FOR WALL RAIL
Art. no. 06502

For attaching to a wall rail. A flexible solution that enables the screen to be moved sideways along the wall rail. Distance between wall rail and screen: 40 mm.



CLAMP WITH SPACER
Art. no. 06535

A useful solution when space is required between a screen and a wall rail, such as in dialysis departments where panel access is vital. Distance between wall rail and screen: 150 mm.



SUPPORT CLAMP
Art. no. 06524

Used with standard clamp 6502 where two wall rails are mounted on the wall.



ADJUSTABLE CLAMP
Art. no. 06516

Adjusts to suit variable spacing between upper and lower wall rails. Clamps can be adjusted between 0 and 150 mm. Used together with standard clamp or clamp with spacer.



BACK SUPPORT ON WALL
Art. no. 06519

Used as backward support when using a clamp mounted on a wall rail. Can be adjusted between 84 mm and 135 mm.

Spacers



SPACER 20 MM
Art. no. 06506

Used to move a screen away from the wall, to make space for a molding for example. Supplied in pairs.



SPACER 100 OR 200 MM
Art. no. 06507, 100 mm
Art. no. 06508, 200 mm

Used to move a screen away from the wall. Supplied in pairs.




ADJUSTABLE SPACER
Art. no. 06509, 200–350 mm
Art. no. 06510, 300–500 mm

Used to move a screen further away from the wall. Supplied individually.

Important measurements

The folding screens can be mounted directly on the wall with spacers or clamps on horizontal medical rails. The preferred method for mounting depends on the type of room. Go to our website for the recommended heights for medical rails. Contact your local distributor to get an optimal solution.

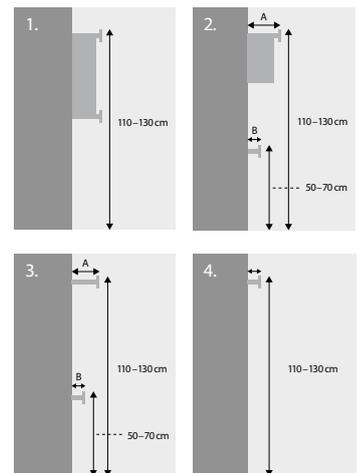

Wall mounting



WALL MOUNTING
Art. no. 06539, height 1.45 m
Art. no. 06544, height 1.65 m
Art. no. 06540, height 1.85 m

For convenient mounting directly to the wall. One wall mounting is normally supplied with the screen. Additional wall mountings can be ordered separately.



WALL MOUNTING WITH FLOOR SUPPORT
Art. no. 06537, height 1.45 m
Art. no. 06545, height 1.65 m
Art. no. 06538, height 1.85 m

Wall mounting that provides support from the floor when fitted with a clamp.



WALL MOUNTING WITH FLOOR SUPPORT AND TOP EXTENSION
Art. no. 06541, height 1.45 m
Art. no. 06546, height 1.65 m
Art. no. 06542, height 1.85 m

Wall mounting used with clamp where a wall rail crosses above the screen.

Floor pole



FLOOR POLE
Art. no. 06561A, height 1.45 m
Art. no. 06562A, height 1.65 m
Art. no. 06563A, height 1.85 m

The floor pole can be used to support a wall-mounted screen or as a mounting point for folding screens. The pole is screwed securely to the floor. Other versions can be seen on page 13.



Colors and designs

A great way to bring life into any patient care setting is by using colors and designs. The Silentia Screen System provides a wide variety of ways in which the screens can be used and combined. Our screens come in a wide range of colors and designs, creating harmony and providing the versatility to blend into any environment.



WHITE




OCEAN



FOREST



SAVANNAH

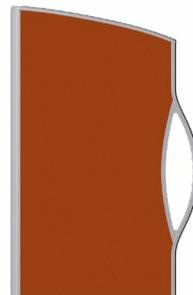

TROPICAL



BIRCH



DAYLIGHT



LIME



MANDARIN




StoryPanels



ArtPanels



PhotoPanels

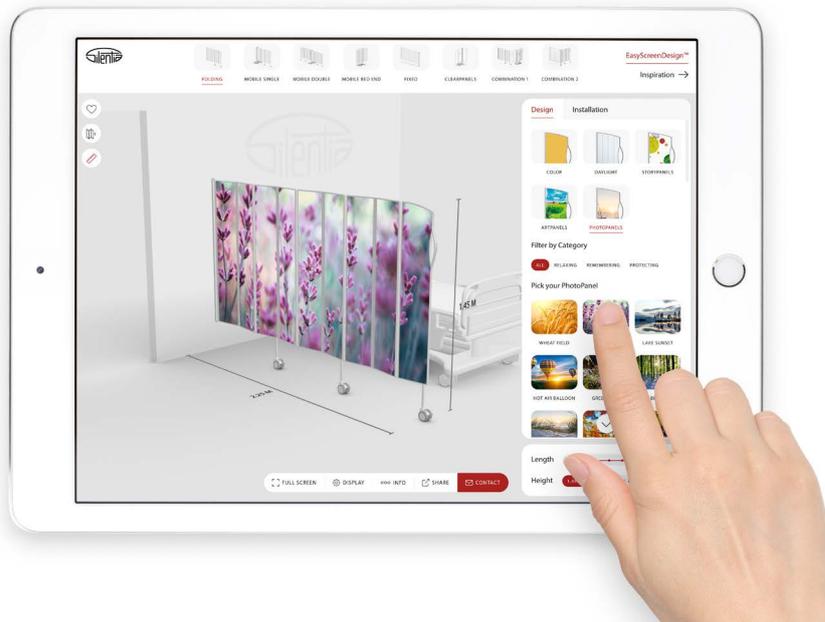

Create your own screen at EasyScreenDesign.com

EasyScreenDesign is Silentia's online configurator that helps you create the exact privacy screen you need for your patient care setting. Pick the type of screen, installation, size and design, then submit them directly to us for a free, no obligation quotation. It's that easy.



EN ISO 9001:2015



EN ISO 14001:2015

Made in Sweden. Our products are copyright-protected in various ways, including registered designs, patents and patents pending. We reserve the right to make changes without prior notice. For more information, please visit our website.

Silentia AB, Box 108, SE-311 22 Falkenberg, Sweden
 Tel. +46 (0)346-485 80, Fax +46 (0)346-485 89
 info@silentia.se, silentiascreen.com

